


## JUST EXTRACT! Dental Extraction Kit

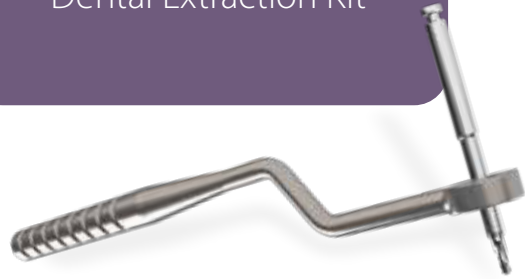

The screw extractor solves stuck/damaged screw problems in:

1. Single Crown.
2. Total prosthesis.
3. Regular/angular abutments.
4. T-Base

A "must have"  
for every dental office



## JUST EXTRACT! Dental Extraction Kit

- Easy, simple and quick extraction process.
- Suitable for standard internal hex connection 3.3mm and 3.75mm
- Maximum Implant protection




2, H'amal Street. P.O.B 1063, Afula 1857107, Israel  
Tel: +972 - 4 - 6094458 | Fax: +972- 4 - 6597812  
info@bh-implants.com | www.bh-implants.com

Design: Shmulik Neto

**Bhi**  
Dental Implants  
The Implant & Implantology Revolution

# JUST EXTRACT! Dental Extraction Kit

# 1

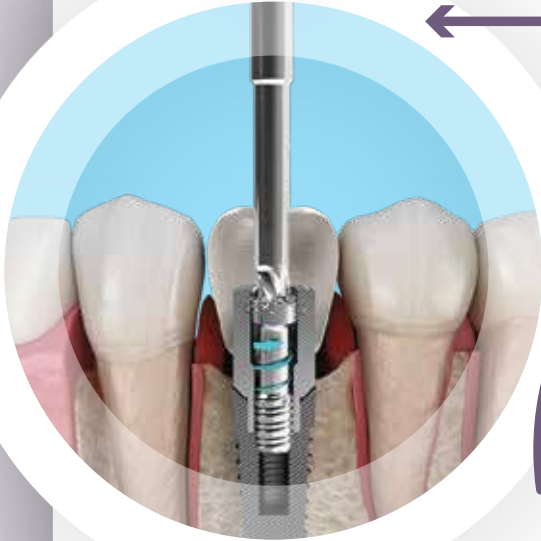


## Procedure 1

Extraction of stuck/damaged screw  
(when screw head is worn)

- The head retriever drill performs two operations simultaneously: one, milling the screw head while rotating it and second, to force it unscrew and climb outside.
- The extraction ends within few minutes. In most cases of extractions, the screw is entirely retrieved.

# 2

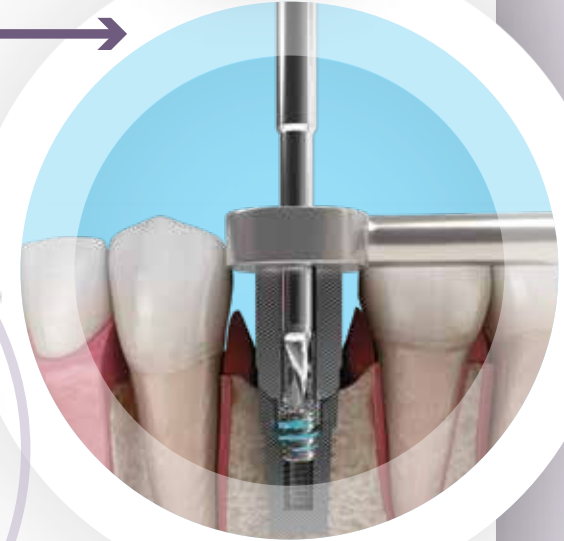

## Procedure 2

Extraction of a broken screw

- The conical drill penetrates the broken screw while unscrewing it and removing it completely.



**A "must have"  
for every dental office**



Patent Pending



Suitable for most internal hex platforms



Superior technology



Reduce chair time



Easy to use

- Patent pending.
- FDA clearance.
- The Extraction Kit must be used only according IFU (instructions for use and safety) which is delivered inside each Kit.



[www.bh-implants.com/ifu](http://www.bh-implants.com/ifu)